

ActionVisc Sodium Hyaluronate

Sodium Hyaluronate 1.8% w/v

(Can also be supplied in 1.4%)



Description:

ActionVisc Sodium Hyaluronate is a sterile, non-pyrogenic, viscoelastic, preparation, of sodium hyaluronate. It contains 18mg/ml of sodium hyaluronate dissolved in physiological sodium chloride phosphate buffer. Sodium Hyaluronate is derived from bacterial fermentation. Its viscosity is responsible for maintenance of space in the anterior chamber and allows the manipulation of tissue during the surgery without escape through the incision. The 1.8% solution is transparent . It does not interfere with epithelization and normal wound healing.

(Other concentrations like 1.4% also available)

Indication:

Indicated for use as a surgical aid in cataract extraction (intra and extra capsular extraction), IOL implant, corneal transplant, glaucoma filtration and retinal detachment surgery.

Contraindication:

Contraindication : At present there are no known contraindications of the use of this sodium hyaluronate.

Warning & Precautions:

- Do not use the package if the integrity of the sterile package is compromised..
- ActionVisc HPMC is for intraocular use only.
- ActionVisc HPMC is for single use only
- ActionVisc is not to be placed in direct sunlight
- Do not resterilize this device.

Storage:

Store at a temperature between 10° Celsius and 30° Celsius and relative humidity less than 65%. Protect from freezing. Do not expose containers to direct sunlight

Supply:

Supplied Sterile in a 1ml Pre Filled Syringe with a 27G cannula

Another Quality Product from:



Action Medical Pvt Ltd.

(An ISO 13485:2016 Certified Company)

Gat No 528, Koregaon-Bhima, Near SBI, Pune-Nagar Road

Taluka Shirur, Dist Pune Pin 412216, INDIA

Tel: 0091-2137-253617 / 18. Fax: 0091-2137-253620.

E-mail : action@actionmedihealth.com

Website: www.actionmedihealth.com

